# Philips Respironics Mask Selector 1.0.0

Troubleshooting and Best Practices

## Introduction

---

The Philips Respironics Mask Selector is a client server application: software with a client installed on a device and used at a facility that communicates with a server in the cloud. The client is where clinicians and patients have all interactions. This client controls both the device camera and depth camera and renders the user interface. The client also communicates with a web server in order to complete things such as logging in, retrieving mask and quiz data, processing 3D images, and grading quiz responses. This communication happens securely over an encrypted internet connection.

This guide should help you get up and running efficiently and troubleshoot common problems. The various sections follow the workflow of the application and provide pointers to fixing issues or figuring out why things went awry, from common setup and installation problems to various error conditions you may encounter.

Thank you again for using the Philips Respironics Mask Selector. We hope you realize all the benefits of using the software for both you and your patients.

Regards,

The Mask Selector Team


## Best Practices

This section covers some of the best practices for using Mask Selector software. These are suggestions to help you get up and running smoothly as well as keep you running at an optimal level.

### Test your internet connection

Since the client transmits up to 2.5MB of data during the entire process, a stable Wi-Fi connection is paramount to having success.

Test your connection using a reputable third-party software tool (an example is shown on the right) available in your device's application store by searching for "network speed analyzer" or "network speed test." However, Philips does not make any representations or warranties regarding the use of any particular third-party application used to test your network and expressly disclaims any liability associated with using a third-party application.

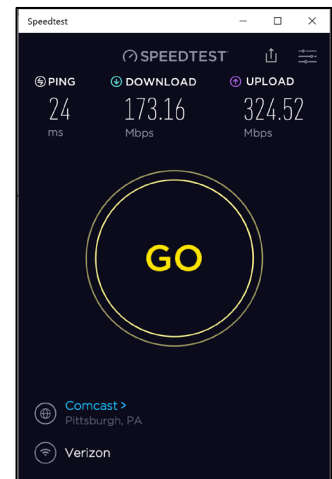The lowest recommended values are 25Mbps for download and upload speeds and less than 100ms for ping. Anything lower than this will result in longer wait times for processing and possibly error messages as timeouts occur.


While this test will ensure you have the necessary throughput, you should also use a third-party Wi-Fi analyzer tool, available on your device's application store, to determine your network crowdedness.

An example analyzer interface is provided to the left. You should ensure that your network occupies a channel that is relatively open. If you see 6 or more networks on a channel it would be best if you move your network to a channel that is furthest from the activity. Most of these settings should be accessible by your IT department if not available on your router. The image at the left illustrates a sufficient gap between channels.

## Ensure Microsoft updates are always current


Windows should be running with the latest builds and patches from Microsoft. These devices are subject to improvements over time, including firmware updates, as shown to the right, that improve power management and USB port communication. Testing will be performed on new updates in order to maintain software performance.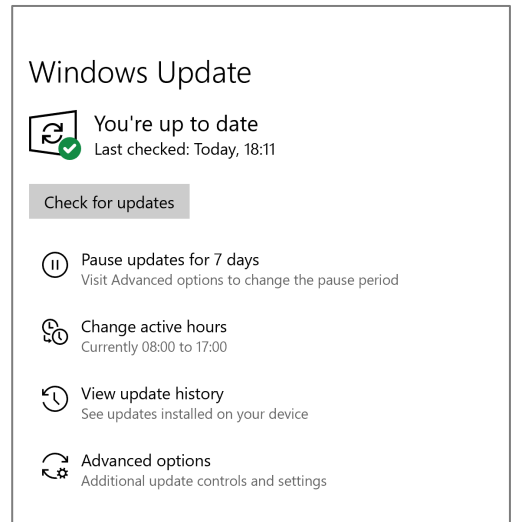

## Maximize power mode for performance

While the application will work with default settings, it works much better when the power mode is set for Best Performance, as shown in the image to the right.

When the power mode is set to lower levels, the device decreases the use of the processor, graphics cards, and other functions to save battery. These power-savings efforts can result in a sluggish scan experience, a failure to build a face, and longer-than-normal processing times.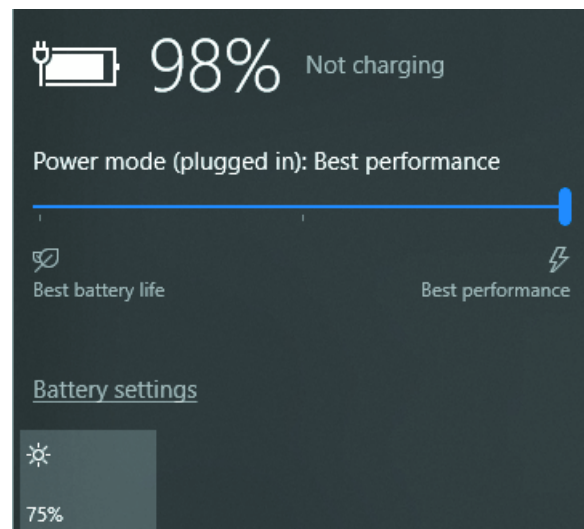

## Reboot your device weekly


While not required, rebooting weekly frees up any long-running processes or reserved resources, ensuring that any background services are running at a proper priority.

## When scanning, remain consistent

When scanning, use a slow consistent sweeping motion. While centering the patient's face, take a few seconds to allow the camera to focus and achieve white balance. You will experience these adjustments as a brightening or dimming of the picture as well as a slight zoom in and out. Once the picture is stabilized, proceed to scan from the patient's nose to the temple. This guideline will ensure you capture 270 degrees of the face including both ears. When making any scanning movement, take 5 seconds to steadily move from one point of the scan to the next. This technique will allow you to get scans that visualize the best on the mask suggestion pages and reduce the number of errors.

## Know where to look when problems occur

All the pertinent logging information is stored in the users AppData directory. This directory is located on the path of `C:\Users\<YOUR_USERNAME>\AppData` and hidden by default. To show the folders, navigate to the path, click the *View* tab and check the *Hidden Items* check box as highlighted in the image below: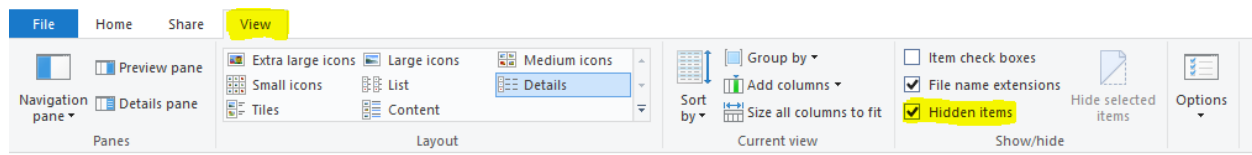

Then navigate to Roaming → PhilipsMaskSelector and find a CMSClient.log file. This file contains any error messages and is vital for troubleshooting problems.

Different errors you may see in this file would include:

```
[2019-05-15T13:16:23] [BellusService] [Error] LocalHeadProcessorObserver::OnError::7, Build 3D face mesh error
```

This error means that you probably scanned too fast or had a unsteady scan. It also pops up if you have the device in power-saving mode and are running a lot of background processes.

```
[2019-05-15T13:16:23] [BellusService] [Error] sourceComponent: HeadProcessor  
errorCode: 13 message: Build 3D face mesh - Face tracking failure
```

If you see this error, you most likely scanned a patient and either exited before completing the scan or started scanning or the patient moved out of the scan area. This error can also happen if the power setting is on power saver and something intensive like a Windows update is being installed.

```
[2019-05-03T10:47:17] [HttpService] [Error] Exception during: HttpGet to  
https://api.maskselector.com/masks/en-US Retry count: 3  
[2019-05-03T10:47:17] [HttpService] [Error] Exception during: HttpGet to  
https://api.maskselector.com/quizzes/en-US/1 Retry count: 3
```

If you see errors like this, you are most likely experiencing network problems such as the network connection dropped, is washed out by multiple overlapping networks, the network is saturated, or the network strength is poor.

```
[2019-05-03T10:50:29] [BellusService] [Error] sourceComponent: DepthCamera  
errorCode: 3 message: DepthCamera is in unknown state
```

If you see this message you may have disconnected the camera by bumping the plug or you never connected the camera at all.

In any case this log will be something that support will ask for and it's something that will begin the troubleshooting process before or during a call to product support.

### **Keep both your camera and the device's camera clean**

If you are experience a "spikey face" (scan that appears to have quills or spikes appearing from the patient's face) or a "smudged face" chances are that your camera is dirty. A lens wipe works to clean both cameras.

### **Keep your device charged when not in use**

Since Windows puts your device in power-saving mode when it hits 20% power it is best to keep your device charged. Having your device in power-saving mode could slow your scans to the point where it feels sluggish and adversely affect processing time.

## Troubleshooting the software installation

---

Sometimes the installation process can fail. Most of these failures are related to network errors, so it is always advisable that you run a CHECKSUM against the downloaded installer to ensure that the download matches the upload. To do this download the QuickSFV utility from here: <https://www.quicksfv.org/qsfv236x64.exe>. You want to download the *MaskSelectorUnifiedInstaller.sfv* and the *MaskSelectorUnifiedInstaller.exe* files to the same directory. Next double-click the SFV file to verify the EXE to ensure it matches the file that was uploaded. If the file fails validation, download the file again. Not completing this check and having a partial file results in errors during installation pointing to failures unpacking files.

Another error may happen if the .NET Framework is not present. In that case, the installer prompts you to download it and you must rerun the installation.

Other errors could be lack of disk space (need to remove files/programs) or elevated permissions (need to see your IT department).

## Problems encountered while viewing application settings

---

In some cases, you may get an error when attempting to view the Settings page. This error is most likely caused by a poor network. The software requests the latest list of masks from the cloud for selection. If that request cannot get through, an error is displayed.

## Problems encountered while viewing the Home page

---

If you are logging in for the first time and get an error shortly after arriving at the home screen, it is most likely a poor network. Much like the Settings screen above, a network "call" is made to retrieve masks. If this network "call" fails for any reason, an error page is displayed.

## Problems encountered while viewing scanning instructions

---

If you receive an error screen on the Scan instructions page, it is the same as above when receiving an error during the Create Patient or Find Patient workflow. It is likely either your camera unplugged or the tracking has timed out.

If you are creating or finding a patient and receive an error message, it is most likely that your camera is not connected. The camera initializes during this screen, creating the odd condition where you are entering data and presented with an error message. Please make sure your camera is connected and try to create or find a patient again.


Another cause of an error on this page is a 10-minute timeout error. Again, due to the camera initializing on this screen, the camera will timeout after 10 minutes in order to preserve battery. If you encounter this error, navigate back to the home screen and create or find patient again.

## Problems before or during scanning


---

If you receive an error while you are scanning, it is likely that the scanner is unplugged, or something is drastically wrong with your installation as one of the underlying libraries may be missing or be corrupted. If this happens, try the scan again. If it fails a second time, check your logs for more info. Another possible failure is if tracking times out. Similar to the two items above, if you start scanning, and then let the software remain idle, you will be prompted to log out. This happens because the software is programmed to save battery. Since the camera is drawing its power from the device, we want to make sure that we only draw the power for a limited time.


If the message "Please wait while we connect your camera" appears for a long period of time and the red light on the 3D camera never appears to be on or blinking, the camera is most likely stuck when initializing. This can happen after Windows updates are installed and the device is awaiting restart. To check if this is happening, tap the Windows icon in the task bar.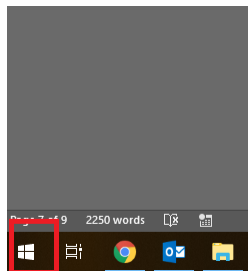

Then tap the gear icon from the start menu.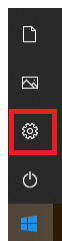


Next tap the Update & Security icon.

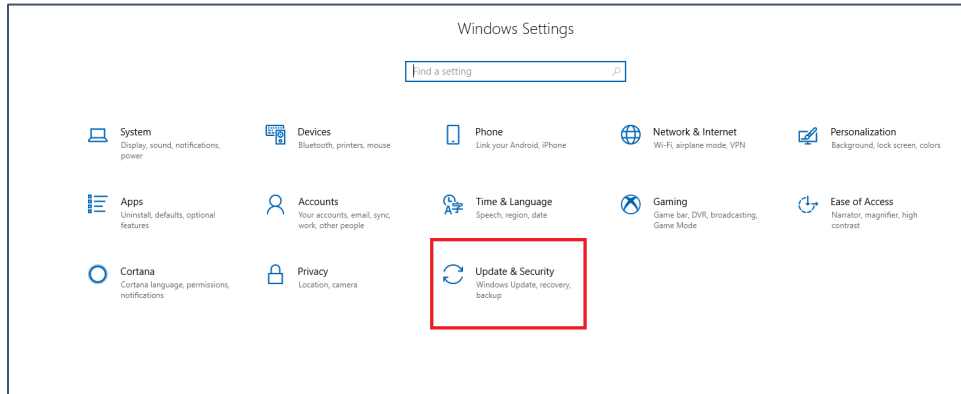Finally check to see if there is a Restart PC button enabled.

Another cause of this could be that the camera failed to connect based on a driver issue. If this is the case, you will need to restart the InuService. Before restarting it, disconnect the camera's USB cable from the device.

Next you will need to long press the Task Bar to open the Task Manager.

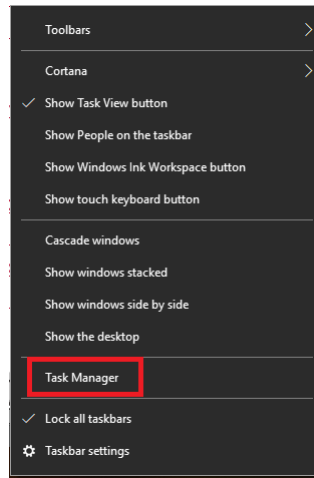
You may then need to select the link titled "More details".

Next tap the details tab at the top of the screen. From the details tab you will need to find the InuService.exe, long press it and select "End Process".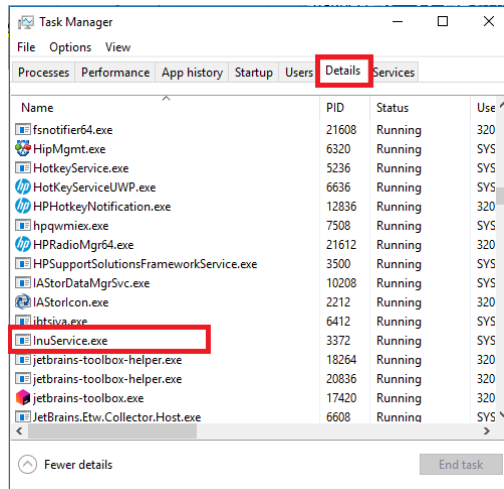

The process should restart in about 10 seconds. After the process reappears in the list of details you should then be able to attempt scanning again.

## Problems while completing the quiz

If, while in the middle of the Quiz, you are presented with an error screen, this could be either your network dropped, or the scan failed processing. What this usually means is that your scan could have dropped frames either by scanning too fast or a processor lag due to power mode settings. Sometimes this scan does get stitched, but it hardly looks like a face and fails processing. In a final case, the scan could be stitched and processed but the result took too long to return and caused an error.

## Problems while on the waiting screen

If you get an error while on the waiting screen it is the exact same case as above with the problems that occur while taking the quiz. It is either an image stitching failure, a scan processing failure, or a network error. You should check your network throughput and ensure you are taking the full 5 seconds to scan each part of the face.

## Need Additional Help?

---

If you need additional help or to have a place of reference for these steps and instructions please see <https://maskselector.com/troubleshooting>.

# Philips Respironics Mask Selector 1.0.0

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